

Specimen collection kit

The kit is used for collection of cells and microbiological specimen from the nasopharyngeal area, followed by storage in the medium for transport to the laboratory for analysis.



Instruction for use

1. Peel off pouch and remove the swab
2. Insert the swab gently to the nasopharyngeal area
3. Remove the cap of the transport tube and place swab into the medium. Either (1) Break the swab along the snapping point and leave the swab inside the tube, or (2) Mix the swab thoroughly in the medium and discard the swab without breaking it
4. Cap the tube and send back for analysis

Product Information

Cat No.	10030
Kit contents	1 flocked swab 1 viral universal transport medium 2 mL
Number of kits	100 per pack
Specimen Type	Upper respiratory nasopharyngeal specimens
Storage	15-30 °C for 12 months